

**NEW!**

# CoFlex® AFD

## ABSORBENT FOAM DRESSING

**LATEX FREE**

**An Easy,  
Cost Effective  
Wound Management  
Solution!**

### All-in-One Foam Pad Dressing and Cohesive Bandage

#### Features:




#### Benefits:

- High absorption and retention of exudate
- Keeps wound site moist and clean
- All-in-one dressing/cohesive bandage
- Soft and comfortable CoFlex LF<sup>2</sup> base
- Waterproof film layer over foam pad
- Expands to conform to wound cavity
- Provides controlled compression
- Easy application
- Non-adherent to wound bed

- Reduces risk of maceration
- Promotes wound closure and natural healing
- Quick - minimizes dressing time
- Improves patient compliance – heals wounds faster
- Prevents leakage, drying out and contamination
- Manages exudate; reduces maceration around wound bed
- Will not constrict over time
- Can be applied by professional or patient
- Protects delicate granulating tissue; avoids trauma during dressing changes

#### CoFlex AFD Construction

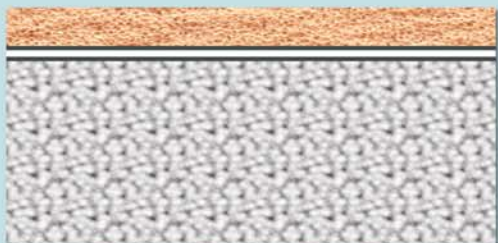

- ← CoFlex LF<sup>2</sup> Layer - Quick stick, cellular foam cohesive bandage is comfortable and lightweight for prolonged wear and increased patient compliance
- ← Waterproof Film - Prevents leakage, drying out and contamination
- ← Absorbent Foam - Foam pad draws in exudate, while keeping wound site moist and clean

#### TESTIMONIALS

“We were very happy with how easy the CoFlex AFD foam pad dressing was to apply. Patients we sent home with the product were able to change the dressings without difficulty and therefore more compliant. We used the CoFlex foam pad dressing on many types of wounds and were pleased with the healing we saw with them all.”

*Amy Mahoney  
Boone Hospital, Wound Clinic  
Columbia, MO*



“The feedback we received from the patients is great and I love the product. Very user friendly.”

*Gwen Mccomy  
Clinical Director  
Jefferson Memorial Hospital Wound Care Center*



# Applications and Uses

- Venous Leg Ulcers
- Arterial Ulcers
- Pressure Ulcers
- Traumatic Wounds
- Diabetic Foot ulcers
- Non-healing Surgical Incisions
- Lower Extremity Wounds
- Skin Tears
- Burns
- Tracheotomies
- Donor Sites
- Chronic Wounds

## General Wound Application



1) Unroll CoFlex AFD to expose the foam pad.



2) Place foam pad on treated, clean wound and wrap bandage at desired tension.



3) Continue wrapping and then seal end of bandage by pressing down and meshing layers together.

## Venous Leg Ulcer Application



1) Unroll and place foam pad on treated, clean wound. Wrap once at desired tension. Hand-tear and seal bandage by pressing down and meshing together.



2) With foot flexed to 90°, continue application beginning with a circular winding of bandage at base of toes.



3) Proceed up and around the ankle, covering the Achilles tendon.



4) Wrap over the ankle and down to secure the heel.



5) Proceed up the leg in a spiral fashion, each turn covering 50% of the previous turn. End just below the knee.

## CoFlex® AFD

ABSORBENT FOAM DRESSING

## ORDERING INFORMATION

U.S. Patent Nos. 5.762.623 & 6.156.424  
 Patent Pending  
 U.S. Patent No. 6.566.576 Dicon Technologies

Catalog#	Size	Rolls/Case
9720TN	2" x 2.5 yds.	16
9740TN	4" x 2.5 yds.	8
9840TN	4" x 7.5 yds.	6
9760TN	6" x 2.5 yds.	8

Sterile (S) = Sterile Product/Tan only.

9720(S)	2" x 2.5 yds.	16
9740(S)	4" x 2.5 yds.	8
9840(S)	4" x 7.5 yds.	6
9760(S)	6" x 2.5 yds.	8



Andover Healthcare, Inc.  
 9 Fanaras Drive, Salisbury, MA 01952  
 Toll Free: 800-432-6686  
 Phone: 978-465-0044  
 Fax: 978-462-0003  
 www.andoverhealthcare.com

