

NEW!


PetFlex[®]•AFD

ABSORBENT FOAM DRESSING

- Foam pad and cohesive bandage in one
- Quick & easy to apply
- Absorbs and holds up to 10 times its weight in fluid
- Strong, flexible PetFlex base
- Controlled compression – will not constrict
- EasyTear[®] technology – no scissors needed
- Effective on puncture wounds, abrasions & lacerations
- Conforms to wound bed - reduces maceration and promotes wound healing
- Won't stick to wound bed

**All-in-One
Wound
Management
System!**

PetFlex AFD Construction

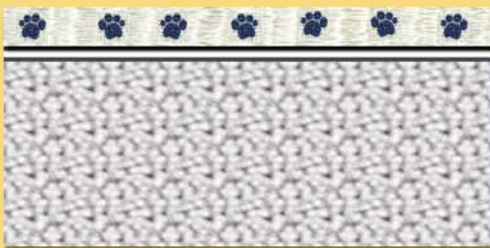

← PetFlex Layer - Strong, flexible PetFlex sticks to itself for quick & easy wrapping

← Waterproof Film - Prevents leakage and contamination

← Absorbent Foam - Patented hydrophilic foam pad draws and holds fluid while keeping wound site clean and protected

A Better Way To Bandage!

PetFlex AFD



All-in-One absorbent foam pad and dressing

Traditional Bandaging



Traditional bandaging method requires a variety of products.

Applications and Uses

Wrapping a leg laceration or puncture wound with PetFlex AFD



1) Once wound has been cleaned, unroll PetFlex AFD to expose absorbent foam pad.



2) Place foam pad over wound.



3) Cover wound with foam pad, wrapping bandage around completely so it covers the first layer.



4) Continue to wrap at desired tension.



5) Seal end of bandage to bottom layer with fingernail for strong, cohesive seal.

ORDERING INFORMATION



Catalog#	Size	Rolls/Case
2220PW	2" x 2.5 yds.	16
2440PW	4" x 2.5 yds.	8

EasyTear
PATENTED TECHNOLOGY



U.S. Patent No. 5.762.623 Patent Pending
U.S. Patent No. 6.566.576 Dicon Technologies



Andover Healthcare, Inc.

9 Fanaras Drive, Salisbury, MA 01952 Toll Free: 800.432.6686 Phone: 978.465.0044 Fax: 978.462.0003

www.andoverhealthcare.com

Proudly Made in USA