

NEW!

PowerFlex® AFD


ABSORBENT FOAM DRESSING

- Foam pad and cohesive bandage in one
- Quick & easy to apply
- Absorbs and holds up to 10 times its weight in fluid
- Strong, flexible PowerFlex base
- Controlled compression – will not constrict
- Straight, hand-tear – no scissors needed
- Effective on puncture wounds, abrasions & lacerations
- Conforms to wound bed - reduces maceration and promotes healing
- Won't stick to wound bed

All-in-One Wound Management System!



PowerFlex AFD Construction

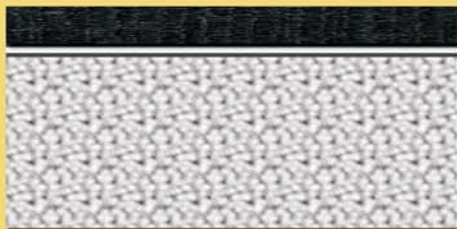

← PowerFlex Layer - Strong, flexible PowerFlex sticks to itself for quick & easy wrapping

← Waterproof Film - Prevents leakage and contamination

← Absorbent Foam - Patented hydrophilic foam pad draws and holds fluid while keeping wound site clean and protected



Andover Healthcare, Inc.
9 Fanaras Drive, Salisbury, MA 01952
www.andoverhealthcare.com

Toll Free: 800.432.6686 Phone: 978.465.0044 Fax: 978.462.0003

Proudly Made in USA

Applications and Uses

Wrapping a leg laceration or puncture wound with PowerFlex AFD



1) Once wound has been cleaned, unroll PowerFlex AFD to expose absorbent foam pad.



2) Place absorbent foam pad over wound.



3) Cover wound with foam pad, wrapping bandage around completely so that it covers the first layer.



4) Prior to each wrap around leg, pull bandage out approximately 6".



5) Continue to wrap at desired tension with 50% overlap.



6) Seal end of bandage to bottom layer with fingernail for strong, solid seal.

Wrapping a neck wound with PowerFlex AFD



1) Once wound has been cleaned unroll PowerFlex AFD, placing absorbent foam pad over wound.



2) Wrap bandage around completely, covering the first layer. Continue wrapping; seal layers of bandage with fingernail (as shown above).

*Photographed at Ever After Mustang Rescue, Biddeford, Maine
www.mustangrescue.org*

ORDERING INFORMATION

Catalog #	Size	Rolls/Case
3744BK	4" x 2.5 yds.	8

U.S. Patent No. 5.762.623 Patent Pending
U.S. Patent No. 6.566.576 Dicon Technologies

